

Nox SAS Solution Setup Instructions



Watch video

1 - Nox SAS Electrode Pack

The instructional video for the Nox SAS Solution hookup can be found on the Nox Medical website: <https://noxmedical.com/sas/>


To begin, clean areas marked with an alcohol* wipe before attaching the electrodes:

- Below both collarbones
- Both sides of lower rib cage

Clean both upper legs on the muscle external to the bone with an alcohol pad - flex the big toes to see the target muscles.

Attach 1x double adhesive electrode sticker to each leg.

*Can be performed with non-alcohol wipes

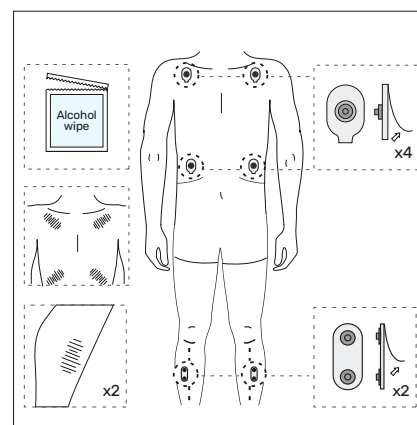


2 - Nox SAS Electrode Pack

Connect the red and blue body cables to the electrodes:

Blue cable on the right side - Red cable on the left side

The single connectors are attached to the single electrodes under the collarbone and rib cage.

The double connectors are attached to the double electrodes on the leg.


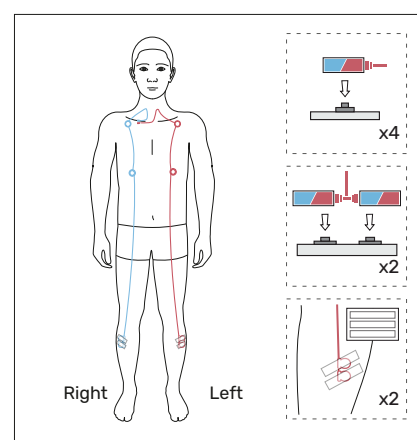


3

Expose the end-connectors of the red and blue body cables by pulling them up through the night shirt neckline.

Attach the Nox A1s with the neck band or with the Nox RIP belts as outlined in the step below.

Attach the red and blue cable to the colour-coded connector attached to the A1s.

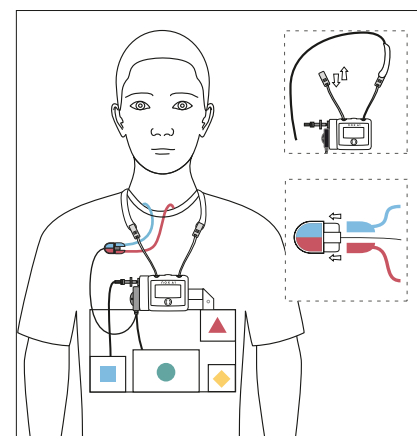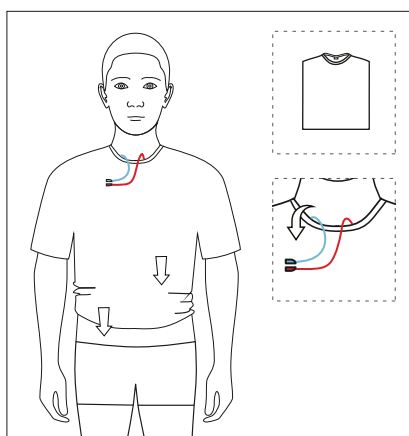


4

▲ Connect 1x Nox RIP belt to the A1s and wrap the belt around the upper chest to connect on the other side of the device.

◆ Connect and wrap the other RIP belt around the abdomen so the connector is positioned on the belly button.

The Nox RIP belts should be untangled and fit snugly without being uncomfortable.

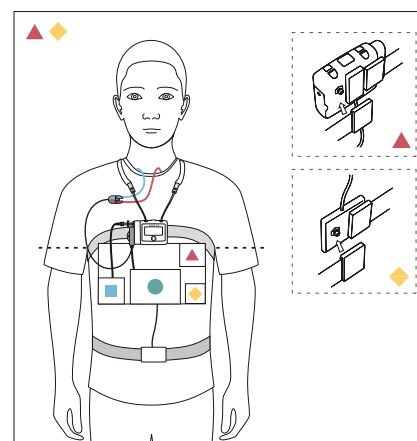

6 - Nox SAS Electrode Pack


Scrub the forehead (down to the cheekbone next to the eyes) and forehead up to the hairline with a wipe.

Place double electrodes in the following positions:

Middle of forehead (horizontally)


Left and right temple at level of the eye (vertically)

Place the last two electrodes between the temple and middle of the forehead snug with the hairline.

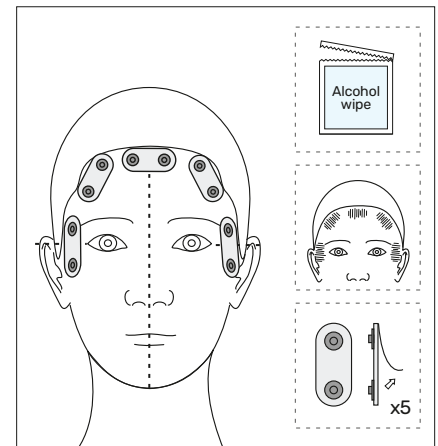


7

Connect the Nox SAS head cable to all the double electrodes on the head. Start with the cable on your right and move to the left side. Use extra tape to secure the head cable if required.

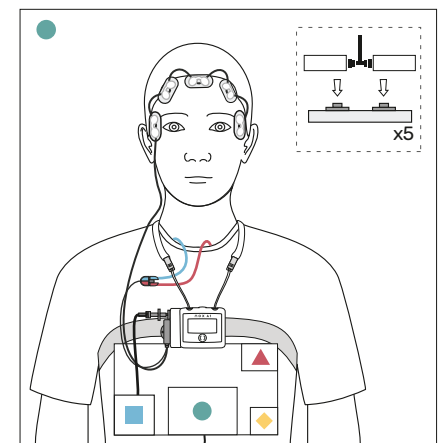

8

Attach the nasal cannula. Place the prongs facing downwards into your nostrils. Tape the cannula in position if required.

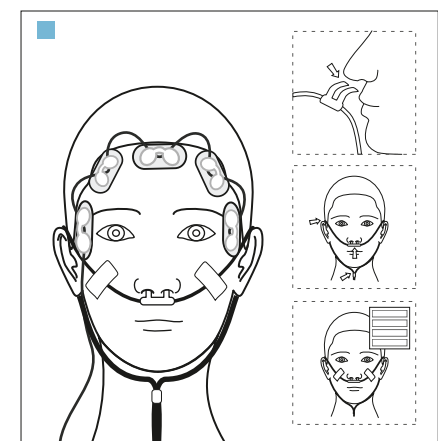

9

Remove the disposable carton from the device by gently pulling it off. Dispose of the carton for paper recycling.

Attach the oximeter and correctly place the probe on the index finger prior to going to bed.

