



Software, Firmware and Manual Updates

Noxturnal 7.2.1 Release Now Available

Noxturnal 7.2.1 is here, bringing powerful new features and workflow enhancements to help clinicians and scorers work more efficiently and confidently.

Key Software Updates:

Improved Titration Management: New titration tools give you a clearer overview and greater control – including PAP traces on the recording results page, oxygen titration support, manual titration support, and full TxLink2 compatibility.

Hypoxic Burden: A novel metric quantifying oxidative stress and cardiovascular risk – now integrated directly in the Recording Results and can be added to reports.

Event visibility in Wake customizable: Choose to hide or gray out events that occur during wake periods for cleaner, more efficient scoring.

Improved Single-Click Scoring: Streamline your workflow with customizable single-click scoring, making event marking faster and more intuitive.

Customizable Parameter Overview: Customize which parameters you see on the Recording Results page – now includes Hypoxic Burden, AHI lateral, AHI supine, AHI REM, AHI NREM, CAI, and RDI for more clinically meaningful insights.

Sentec tCOM+ Integration: Support for the new Sentec tCOM+ device for in-lab recording.

Integration with Philips LoFlo™: Seamless integration with Philips LoFlo capnography system, enabling continuous end-tidal CO₂ monitoring and waveform analysis directly within the Nox A1® PSG workflow for enhanced respiratory assessment.

Numerous bug fixes, including PAP table refinement.

Firmware & Manual Updates

Updated firmware for Nox T3s and Nox A1s (v3.1.2.2868) with pairing improvements, stability fixes, and the option to turn on/off the successful study indicator.

This release includes updated manuals for the Nox T3s and A1s, now conveniently available on the Nox Medical support website instead of embedded in the software.

[Download Noxturnal 7.2.1](#)

Downloading the Software

The latest software, firmware, and updated language packs are available on the Nox Support Site.

- [Noxturnal 7.2.1](#)
- [Noxturnal Language Pack](#)
- [Noxturnal US 7.2.1](#)
- [Nox T3s & A1s Firmware](#)

Learn More: For the full list of features, improvements, and upgrade instructions, check out the complete Noxturnal 7.2.1 Release Bulletin

[Noxturnal 7.2.1 Release Bulletin](#)